


TAP PAP CS Connector Parts

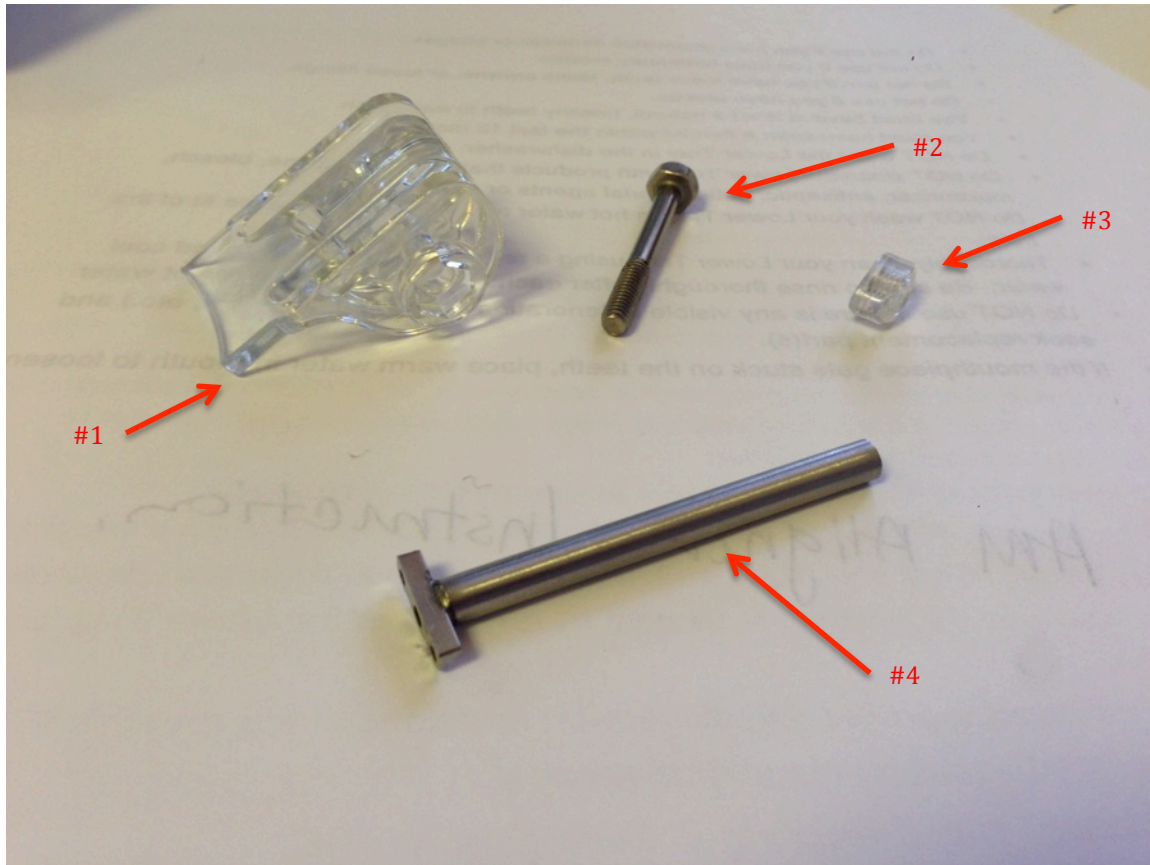

1. TAP PAP CS Connector
2. Screw
3. Nut
4. Front




Assembling the TAP PAP CS Front Plate with Post to the TAP 3 Elite


1. Disassemble the TAP PAP CS Connector. Set the parts aside for later on.



2. Remove the TAP 3 Elite Front Plate by removing the 2 Locking Screws (outer screws).

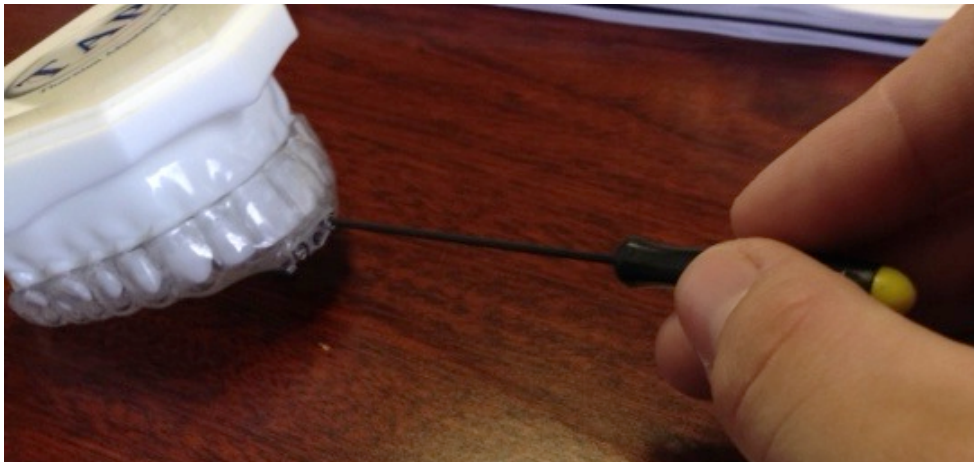


3. Remove the front plate, leaving the hook attached. (Set the TAP 3 Elite Front Plate aside, this will no longer be needed)

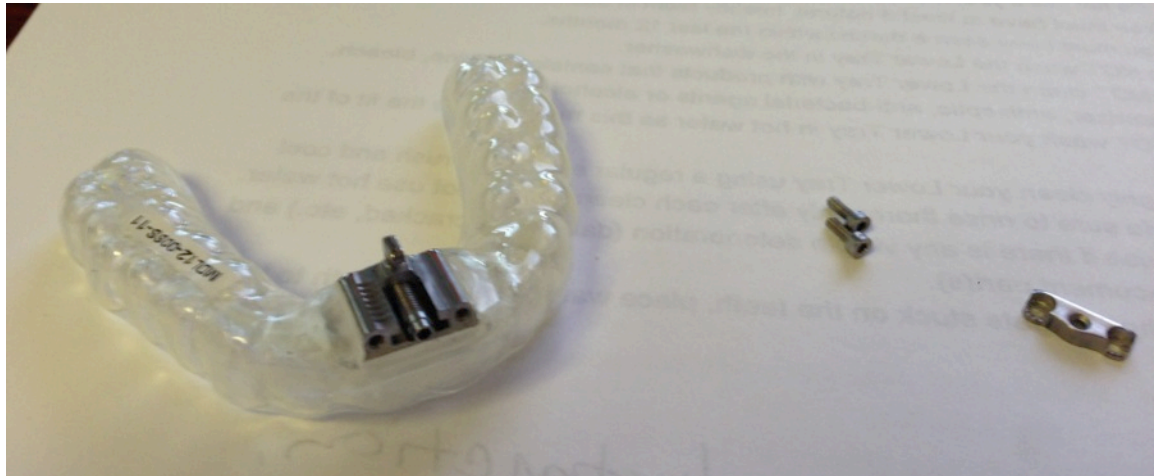

4. Attach the Front Plate with Post from the disassembled connector onto the TAP 3 Elite (where you removed the front plate)
Note: Be sure the front plate goes on with the flat side of the front plate post facing up.



5. Using the Locking Screws you removed from the TAP 3 Elite in step 2, secure the Front Plate with Post to the TAP 3 Elite Upper Appliance.


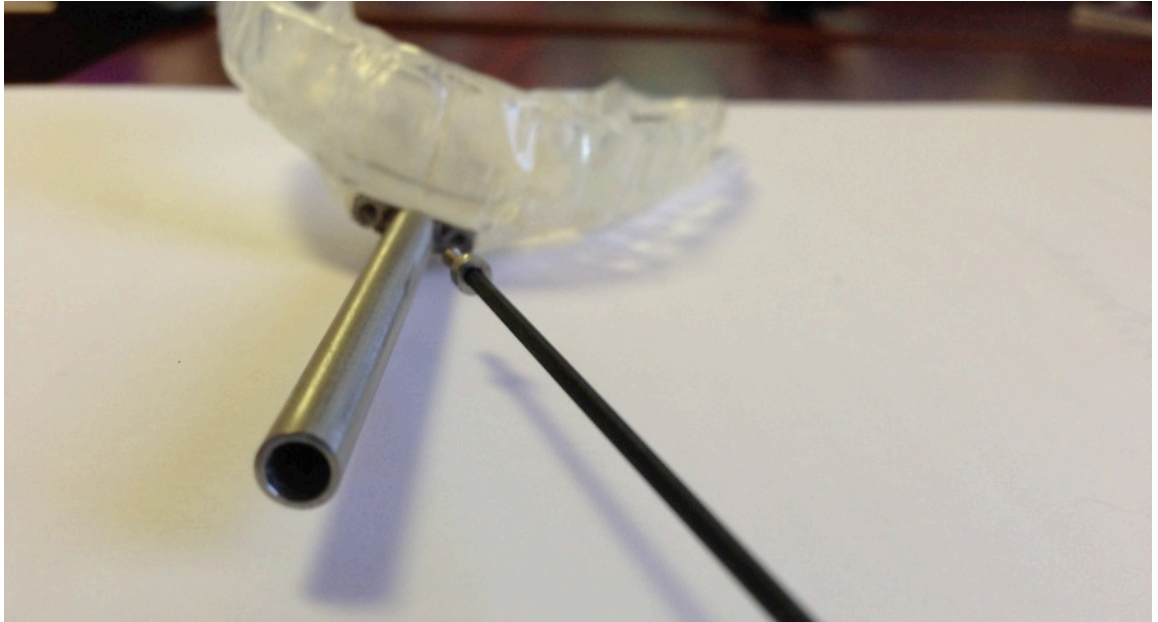


Assembling the TAP PAP CS Connector to the TAP PAP Nasal Pillow Mask Frame


1. Slide the TAP PAP Nasal Pillow Mask Frame onto the TAP PAP CS Connector.
Note: Be sure the flat surface is facing upwards (as shown).




2. Insert the TAP RAPS Connector Nut and Screw into the Connector and Mask Frame.

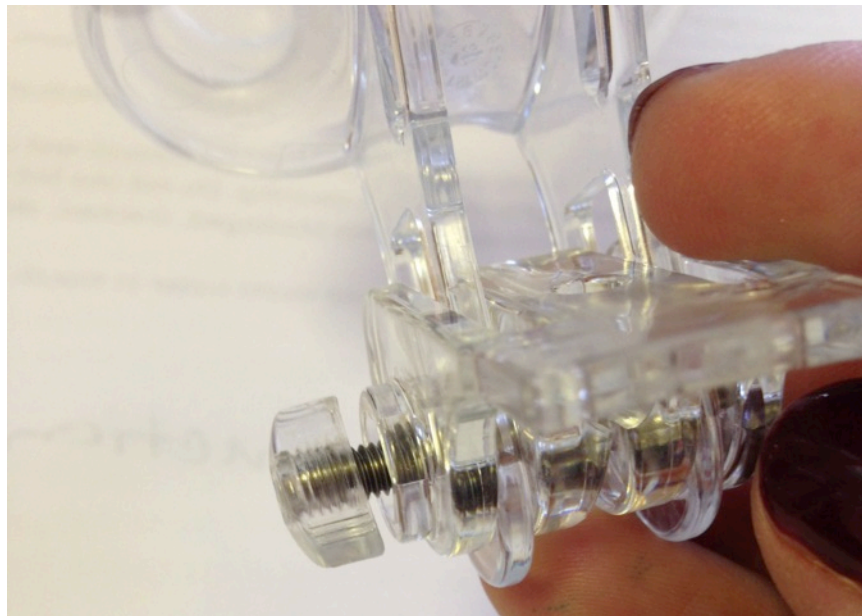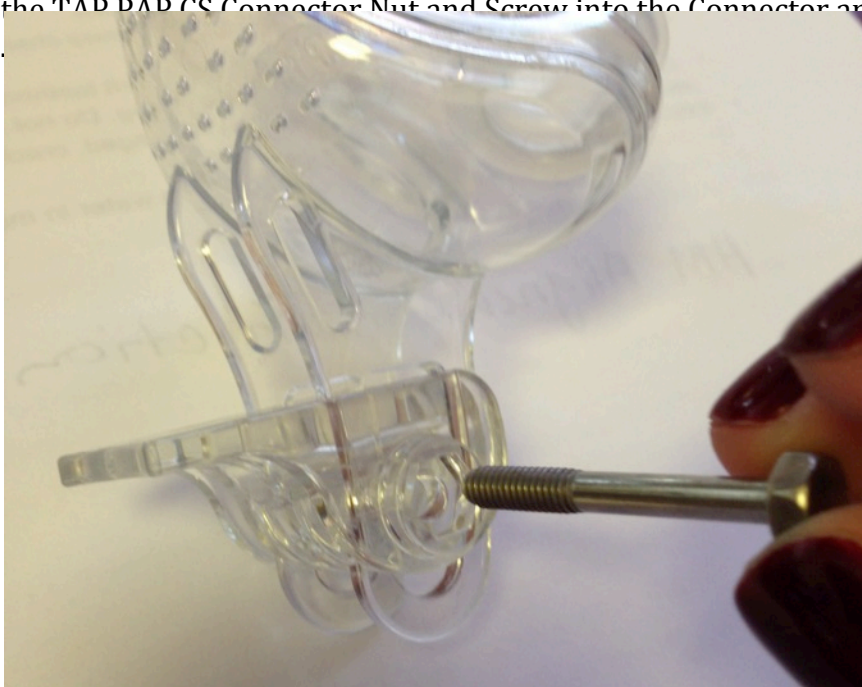

3. Insert the Connector Post (attached to the TAP 3 Elite Appliance) into the Connector (attached to the TAP PAP Nasal Pillow Mask Frame).

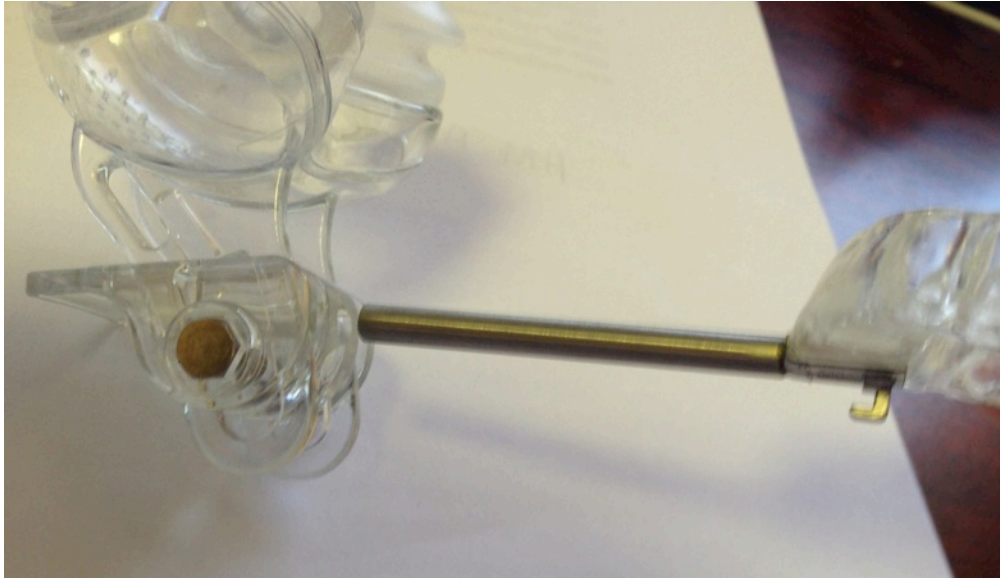

4. Once the Connector Post is attached to the Connector and Mask Frame, lightly tighten the Screw and Nut to hold the post in place.

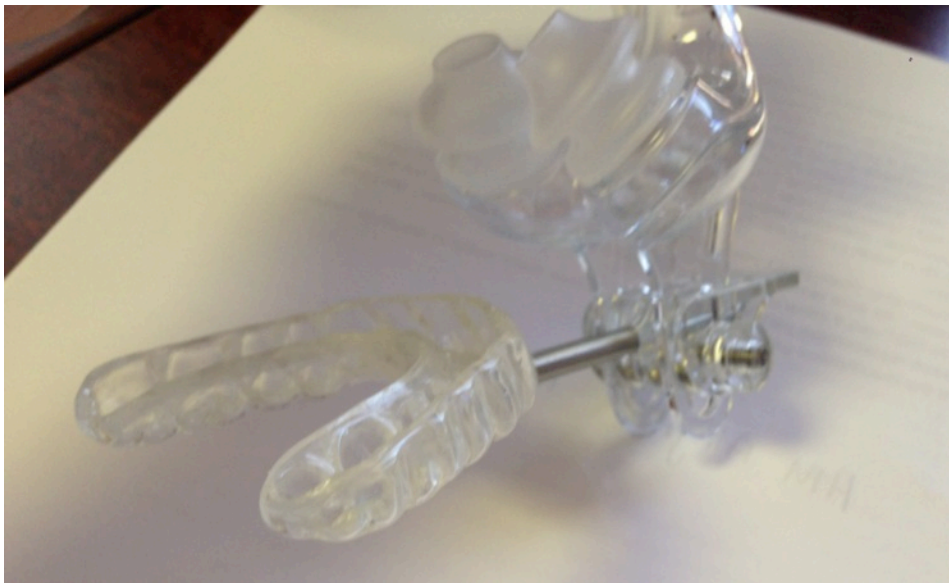

5. Your TAP PAP CS Connector is fully assembled. Refer to the TAP PAP Nasal Pillow Mask Frame Instructions for fitting the Mask.

